



8000DY QUICK INSTALLATION GUIDE


- 1** After removing the box from the pallet. Remove the pump cover via 3 Philips screws.

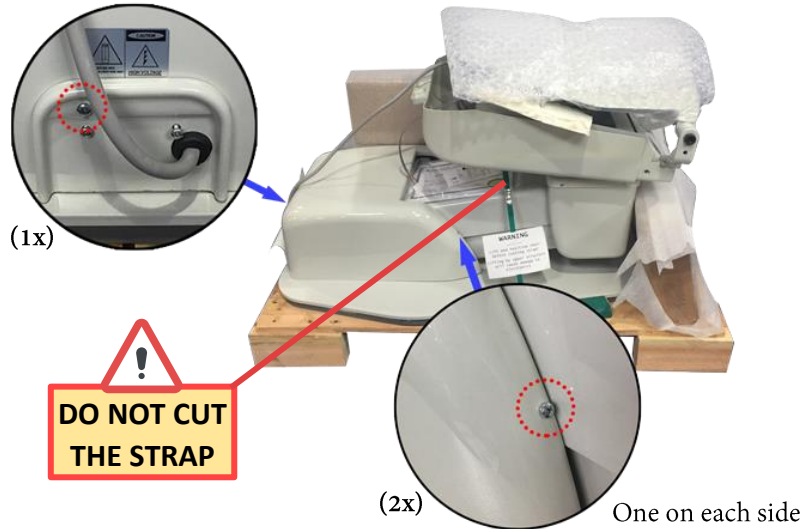


- 2** Using a ½” wrench, remove shipping bolts. Remove shipping blocks.

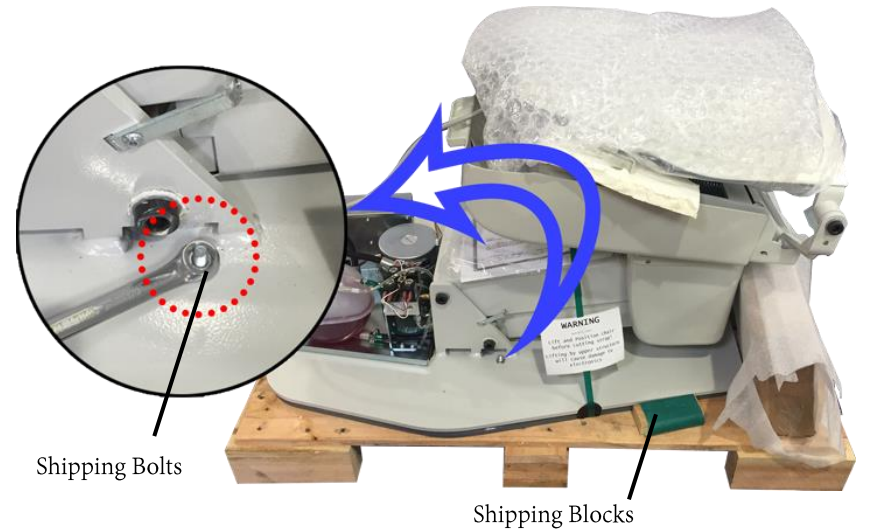

- 3** Lift the chair off the pallet using the backrest mounting brackets.

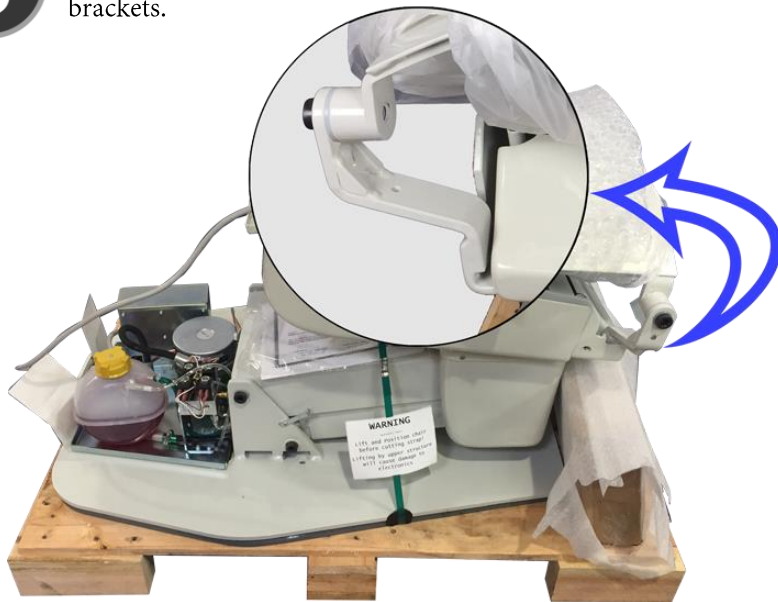


- 4** Once the chair is positioned in its final location, cut the strap and remove the shipping screw from the reservoir cap.

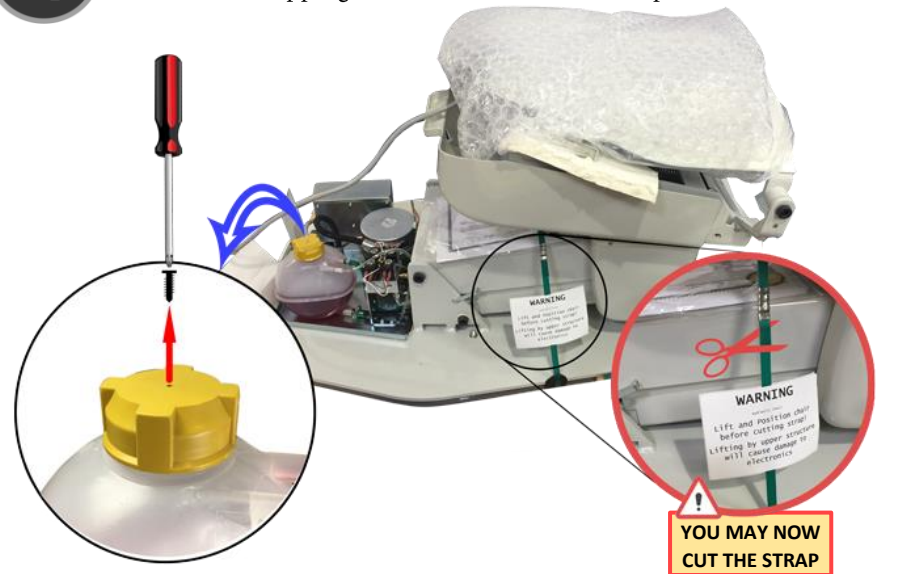

8000DY QUICK INSTALLATION GUIDE

5 Raise base to full up position, lift backrest, install pin, and using a 1/8" Allen wrench tighten set screw.


Make sure the membrane harness is behind the bracket.

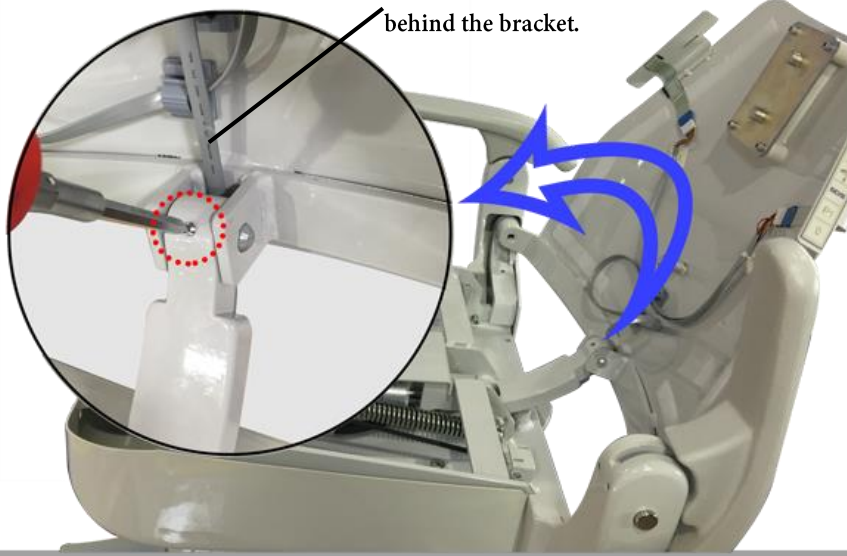

6 Take the back piston cover off by removing the two screws located underneath.



7 Install armrest and toe pan using 1/4" and 3/16" Allen wrenches.

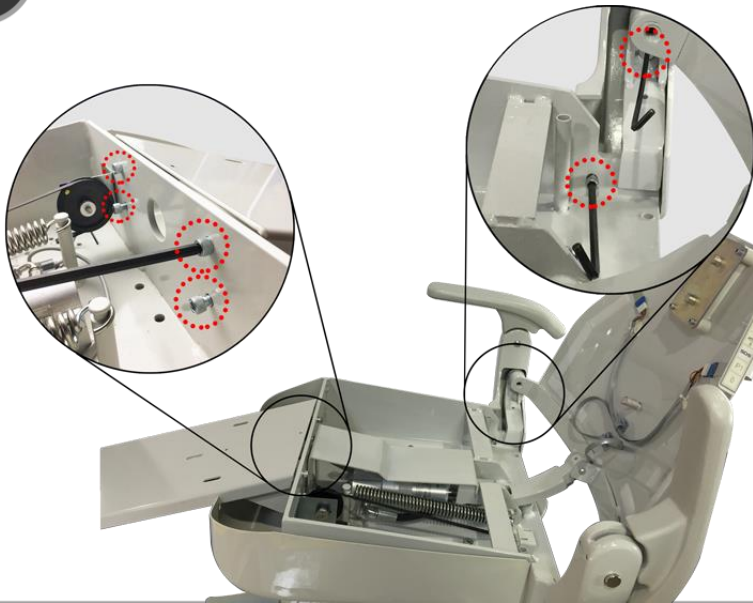

8 Install upholstery, re-install all covers, and test all chair functions.

